



LibertyBOSS

## Annual Sludge Verification Report

### **How to inspect the sludge level in a LibertyBoss amalgam separator**

- 1) Ensure that the pump system has been turned off for 6 to 8 hours.
- 2) Identify the inlet side of the separator (directly below the inlet for on the blue dome at the top of the unit).
- 3) Below the blue upper dome section, on the inlet side, is the Sedimentation chamber of the separator.
- 4) You must use an LED light and aim it back into the sedimentation chambers. To do this, hold the light on the corner of the sedimentation section where the front and back sections split.
- 5) Note that you will be looking for the line of light grey material.
- 6) Take the reading and mark the volume on the Separator Est. Sludge Weight at the bottom of the Annual Sludge Verification Report.
- 7) Complete all the sections of the Annual Sludge Verification Report and send by Fax to 1-905-723-9610 or email to [info@marsbiomed.com](mailto:info@marsbiomed.com).
- 8) Keep a copy in your records.