



**MARS**<sup>TM</sup>  
BIO-MED PROCESSES INC

**CLEANLINZ**  
CONCENTRATE

## Quarterly Dental Evacuation Line Cleaning System

Super concentrated formula digests organic matter & eliminates odors while restoring systems suction to full capacity.



- All natural bacteria & enzymes
- Reduces the population of H<sub>2</sub>S forming bacteria
- 100% Natural & Biodegradable
- Environmentally safe
- Eliminates sludge & reduces odor
- Safe for all animals, plants & pipes
- Non-foaming and safe to use for all pump systems
- Provides optimized degradation of organic waste

## Instructions:

- To be used at the end of the day for 8 days every 3 months.
- For 2 Chairs: Add 1/2 oz (15 ml) of the CleanLINZ concentrate to 32 oz (1 litre) of warm water.
- Mix well and suction into each evacuation line at the end of the day.
- Turn off the suction and allow CleanLINZ to work overnight.

## Contact Us:

Toll-free:  
Canada & U.S.A. 1-866-594-3648

MARS Bio-Med Head Office:  
1801 Wentworth Street, Unit 1  
Whitby, Ontario L1N 8M2  
F. 905-723-9610  
E. info@marsbiomed.com

**1 x 32 Ounce (1 Litre) bottle will last 4 chairs 1 year or 128 applications.**

[www.marsbiomed.com](http://www.marsbiomed.com)